

Fabric Isolation Gowns

Fabric/Washable Isolation Gowns are now available

These gowns are used in a similar manner to the disposable gowns **EXCEPT** after use,

- **Disposable gowns** – discard in waste bin (one time use then throw away)
- **Fabric/Washable Gowns** – place in designated hamper/soiled utility for laundering (one time use then send for laundering)

Do not hang or store gowns that have been removed.

The blue or yellow washable gowns will be stocked on the clean linen carts on the unit.

Donning: Unfold the gown and hold it so the opening is toward your back. Place your arms through the sleeves. Tie the gown snugly at the neck and waist

Doffing: Perform hand hygiene – with clean hands - Remove the gown by pulling it off from the neckline, so that the sleeves end up turned inside out. Ball the gown and place it into the designated hamper.

