

Donning



Begin in a sitting position, remove footwear, tuck scrubs legs into the top of your socks. Apply suit, one leg at a time, put shoes back on. Bring suit up to the waist. Stand up and continue to pull suit up, one arm at a time.



Apply the appropriate N-95 and eyewear protection. Pull hood of suit up and over your head and zipper the suit up under your chin.




Apply gloves ensuring that the end of the gloves come up and over the cuff of the suit. Perform a simple stretch or knee bend to ensure adequate room to maneuver in the suit.




Doffing



With gloved hands only touching the suit material: Unzip suit to the waist and lower hood—mask and goggles remain in place.

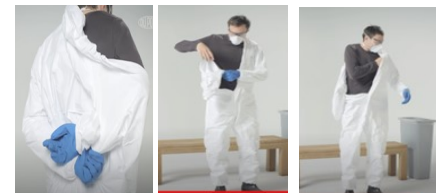

Place hands behind your back and pull the suit off one shoulder. Continue to pull the one arm free, removing glove simultaneously with the sleeve. Take your newly ungloved hand and slide it inside the clean part of your suit and draw the suit over and down your other arm, removing the glove with your sleeve.

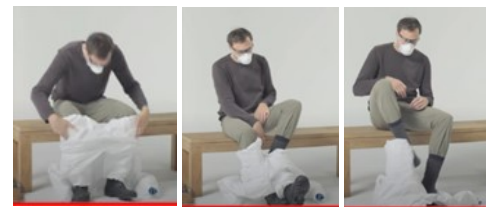

Begin rolling your suit it on itself as you sit down and roll your suit to knee level. Make certain that your ungloved hands are only touching the inside of the suit. Remove one foot at a time. Roll suit into a ball and discard in appropriate receptacle. Remove/discard goggles and mask. Perform hand hygiene.

Tyvek Suits can be worn in place of isolation gowns when caring for COVID -19 patients

Ordering

- Sizes: M, L, XL, 2XL
- Follow current PPE ordering process from materials management.